


Allevyn Gentle Border Lite: Use for a scant amount of exudate.

Application

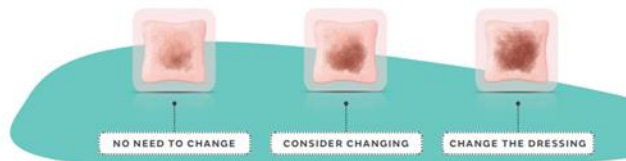

Indicated for up to 7 days or when exudate reaches within 0.5 cm from the edge of dressing.

Formulary Sizes:

5cm x 5cm (absorbent pad size 3cm x 3cm), 10cm x 10cm (absorbent pad size 7.5cm x 7.5cm), 15cm x 15cm (absorbent pad size 12.5cm x 12.5cm).

Allevyn Gentle Border: Use for low to moderate amount of exudate or skin tears, or where dressing will be changed frequently.

Application





Indicated for up to 7 days or when exudate reaches within 0.5 cm from the edge of dressing.

Formulary Sizes:

10cm x 10cm (absorbent pad size 7.5cm x 7.5cm), 12.5cm x 12.5cm (absorbent pad size 10cm x 10cm), 17.1cm x 17.9cm (absorbent pad size 11.6cm x 12.3cm).

Allevyn Life: Use for moderate to large amount of exudate or skin tears.

Application

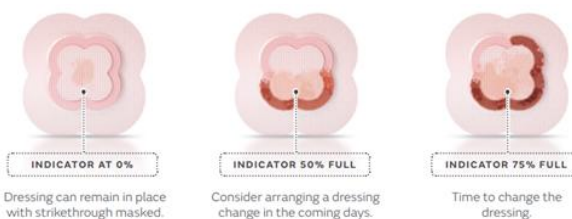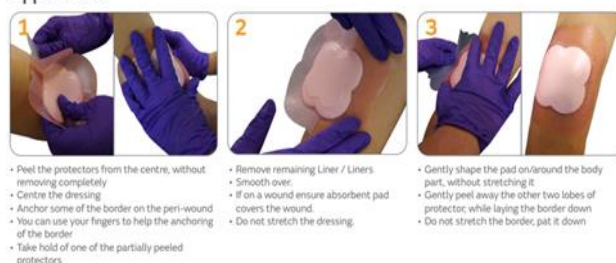

Indicated for up to 7 days or when exudate is visible in 3 out of 4 lobes.

Formulary Sizes:

12.9cm x 12.9cm (absorbent pad size 7.6cm x 7.6cm), 15.4cm x 15.4cm (absorbent pad size 10.2cm x 10.2cm), 21cm x 21cm (absorbent pad size 15.4cm x 15.4cm).

Allevyn Non-Border: Use for a moderate amount of exudate, requires secondary fixation.

Application



Indicated for up to 7 days or when exudate reaches within 1.5 cm of dressing edge.

Formulary Sizes:

10cm x 10cm, 15cm x 15cm.

Allevyn Adhesive Classic: Use for a moderate amount of exudate.

Application



Indicated for up to 7 days or when exudate reaches within 0.5 cm from the edge of dressing.

Formulary Sizes:

12.5cm x 5cm (absorbent pad size 10cm x 10cm), 17.5cm x 17.5cm (absorbent pad size 15cm x 15cm).

Removal:

To remove the dressing, lift one corner of the dressing and slowly peel back until completely removed from the wound.

Do **not** use foam on feet!

Poster Link: <https://www.healthassociation.ns.ca/wp-content/uploads/2023/03/No-Foam-Zone-feb-2022.pdf>

Foams are **not** to be used for prevention!